



Fitting Instructions



Not for braces - Not therapeutic

After fitting, Imako® teeth are arch-shaped and springy, with your tooth imprints on the back. It stays in by having shapes which match the outer surfaces of your teeth, not sticky adhesive.



So thin, light comes through if fitted properly!

Items needed: Mirror, timer with seconds, pot of scalding water, ceramic bowl and nail clippers.

The back layer, the white beads and the pink chunk are the same material, which is soft when hot and hard when cool. It will bond to itself when a hot surface touches a hot surface.

1. Wash hands, floss and brush teeth.

2. Soften the chunk of pink plastic in a bowl of scalding water. Remove with a spoon. Roll it out on your palm until you have a thin tube like a spaghetti noodle as wide as your palm (you have more than necessary). Keep soft by dipping in hot water.



3. Turn the Imako® teeth upside down and briefly dip halfway into the hot water to make it receptive to bonding the pink strip.



4. Attach the pink spaghetti piece to the top edge of the back layer. This will form to hug your gum and give the Imako® teeth a strong 'memory' of your exact arch. It forms a ramp at the top so your lip won't catch on it.

5. Refresh scalding water. Put teeth face down in the bowl. When back layer is completely clear, pour off water. Start timer. Wait 10 seconds.

6. Looking in the mirror. Pull lip forward, insert and position teeth. Press very firmly against each of your teeth, then hold it absolutely still. Press gently through your lip to make sure pink plastic is resting on your gum.

Use one hand to hold the sides in place and the other hand to keep the front from moving. Keep still until two minutes on your timer.



7. Remove teeth by spreading from below. Don't touch top. Let cool on counter. Try on! If not satisfied put back in hot water, scrape plastic up from bottom halfway. Remelt, try again.

8. Snip off the tips of the points formed where two teeth meet the gum; They can cause discomfort.



Do not allow Imako® to completely set in your mouth! If device is stuck DO NOT USE FORCE! Sip very hot water, lean face down, hold water against plastic gripping your teeth. Scrape and pinch away plastic, bit by bit. Plastic melts at 125°. Your mouth can withstand 185°.

Coating Technique to Resist Staining and Add Shine

After fitting, brush with detergent & allow to dry completely. Paint the front layer with a thick coat of clear nail polish. Do not overwork (keep brushing) the polish. Let dry overnight. No allergic reactions have ever been reported.

Cleaning & Maintenance

Eating is not recommended. Avoid high-staining red wine, mustard, curry, tobacco, etc. Brush with toothpaste. Wipe with 91% rubbing alcohol and soak in peroxide every night. The strongest cleaner available is 40% peroxide from the beauty supply store.

How to Improve the Appearance

You can trim the i-teeth with nail or cuticle clippers. Round off with a nail file.

Missing Some Teeth?

The white beads are used to make projections on the inside surface which fit between your teeth. These projections work like the male side of a snap fastener, and reinforce the strength where your teeth are missing.

Melt two beads per missing tooth. Dip the area where you have no tooth print very briefly in hot water (to make it receptive to bonding). Place beads there. Position in mouth and from behind, press the warm plastic to butt up to neighboring teeth, but not behind them. Let cool until material is slightly soft and remove.

It's easy to add more but hard to take away so be conservative.



For teeth missing on one side, use melted beads to wrap halfway around the last tooth. To wear, put this side on first and take it off last.

Guarantee

Return to seller if their policy allows. If purchased direct, the Imako® Cosmetic Teeth can be returned within 30 days. If unused we issue a full refund minus shipping cost. Imako® teeth come inside a sealed plastic sleeve, which if opened means the item is "used". \$19.95 is refunded for used teeth. Place used teeth in a plastic bag before mailing. Don't forget to include your info.

Swapping

No matter where you purchased, swapping is free if teeth are not used. In a regular envelope, mail only the teeth and keep the kit. A used teeth swap costs \$15. Flatten used teeth in hot water, put in plastic bag and mail in a regular envelope.

Include your name, address, telephone, email and what you desire. Include payment or we will email invoice for used teeth swaps using a credit card

Visit our tutorial: goo.gl/JFDOMD or call for voice instructions-press option 1

Dr. Bukk, Inc.
704 S. Old Belair Rd.
Grovetown, GA 30813
866-513-7308 or 706-855-1514
<http://www.imako.net>
Hours: 10-5:30 M-F EST