





FITTING INSTRUCTIONS

NOT FOR BRACES OR PARTIALS.
NOT A THERAPEUTIC DEVICE.

VISIT OUR TUTORIAL: IMAKO.COM/FITTING-VIDEO

After fitting, **IMAKO**® teeth are arch-shaped and springy. Projections are formed which fit into depressions where two teeth meet your gums; This is how they stay on!

THE PERFECT FIT!

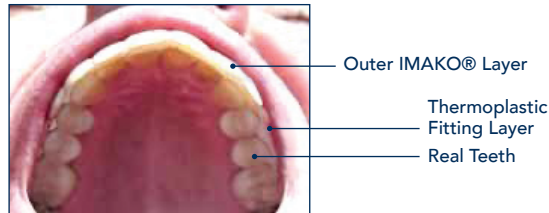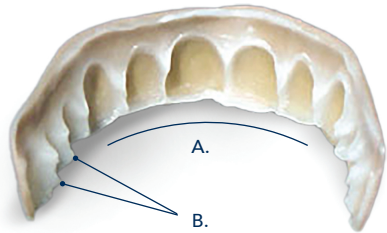

How it Fits - Fitting Material creates exact impression of your teeth. **IMAKO**® teeth must be positioned slightly lower than your real teeth. **IMAKO**® snaps on without adhesive.

A) Tiny ledge under front bite provides spring tension to device, and helps stabilize position.

B) Points between teeth lock device onto your teeth.

Kit includes: **IMAKO**® Teeth, Fitting Beads, Gum-colored Thermoplastic, and Carrying Case.

Items needed: A Mirror, timer, hot water, spoon or fork, ceramic bowl or glass, and nail clippers. The back layer, the white beads and the pink chunk are made of the same material, which turns soft when hot and hard when cool.

1. Wash your hands, brush and floss your teeth. Remove lipstick (if applicable).

Optional: Perform steps 2-4 for additional gum support.

2. Soften the pink plastic in a bowl of hot water. Remove with a spoon. Roll it out on your palm until you have a thin tube like a spaghetti noodle as wide as your palm. Keep soft by dipping in hot water.

3. Turn the **IMAKO**® teeth upside down and briefly dip halfway into the hot water so it will bond to the pink strip.



4. Attach the pink spaghetti piece to the top edge of the back layer.

5. Refresh hot water. Place teeth face down in the bowl until the back layer is completely clear. Once clear, take the teeth out and **Start timer for 2 minutes**

6. Look in the mirror. Pull your lip forward, insert one side then the other. Center **IMAKO**® teeth by looking at your nose. Check the height of the sides to ensure that your natural teeth are not showing. Press and pinch firmly against each tooth. Press through your upper lip to thin the pink plastic. Use one hand to hold each side in place and the other hand to keep the front from moving. Hold the position for 2 minutes and no longer.

7. After 2 minutes, remove the teeth one side at a time Don't touch the top! Let cool on the counter and try them on! If not satisfied, put back in hot water, scrape plastic back to the original position, remelt and try again.

8. To prevent discomfort, use nail clippers or scissors to trim the tips of any points formed where two teeth meet the gum. If there is any visible plastic below the tooth line, dip the area in hot water and pinch away the excess plastic with your fingers.

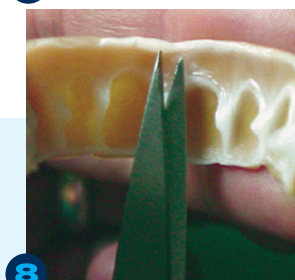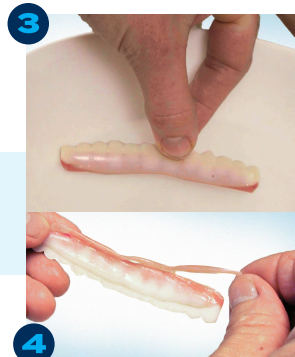

Do not allow **IMAKO**® to completely set in your mouth! If device is stuck **DO NOT USE FORCE!** In order to remove the teeth, take a mouth full of hot water and tilt your face toward the ground, insuring that the **IMAKO**® teeth are fully engulfed by the hot water. The plastic will begin to soften when placed in contact with the hot water. Be sure to test the water temperature prior to placing it in your mouth to avoid burns.

Cleaning & Maintenance

Eating is not recommended while wearing. Avoid smoking and high-dye content foods and liquids such as red wine, coffee, mustard, curry, tobacco, lipstick, etc. To clean, soak the **IMAKO**® teeth in peroxide overnight. Thoroughly rinse the teeth of any remaining peroxide prior to the next use.

Missing Some Teeth?

Optional: In order to fill any major gaps in your teeth, use **IMAKO**® **Moldable White Fitting Beads** to make projections on the inside surface of the **IMAKO**® teeth where the gaps are located.

Melt two beads per missing tooth. Dip the area where you have a missing tooth very briefly in hot water and place beads there. Place the **IMAKO**® teeth back in your mouth and from behind, press the warm plastic to butt up to your neighboring teeth, but not behind them. Let cool until the material is slightly hard and remove.

Be conservative, it's easy to add more but hard to take away. For teeth missing on one side, use melted beads to wrap halfway around the last tooth. To wear, put this side on first and take it off last.



Guarantee

Return to the seller if their policy allows. If purchased directly from **IMAKO**®, the **IMAKO**® Cosmetic Teeth can be returned within 30 days from the date of purchase. **IMAKO**® will issue a full refund minus shipping for the timely return of any unused teeth in the original packaging with an unbroken seal. If the plastic seal is broken the teeth are "used". We refund \$19 for used teeth. Seal used teeth in a plastic bag and include your info.

Swapping - Include Your Info

No matter where you purchased, swapping is free if teeth are not used. In a regular envelope, mail only the teeth and fitting material, **keep the kit**. A used teeth swap costs \$15. Flatten used teeth in hot water, seal in a plastic bag and mail in a regular envelope.

Include your name, address, telephone, email and what you desire. Include payment or we will email invoice payable by credit card.